

1

Keep Chair in vertical position and seat the passenger.



Tilt chair back and push gently over top step. Skids must be in contact with Two steps to descend. Always keep a firm grip on the chair.



3

Gauge your spread. When wheels touch landing, repeat for next flight of steps or deploy support wheels and move to safety.



4



2

Deploy rear support wheels. Fasten seat belt and head restraint. Move towards stairs. When at stairs, close rear support wheels and lock safety in place.

5



NO

Do not let the chair run out of landing



YES

Pivot turn now with weight directly over axel



Distributed by:



Templeacroha  
Clonroche  
Enniscorthy  
Co. Wexford

051 420 575

barronlifts@icloud.com

www.barronlifts.ie



Registered in the Rep. of Ireland